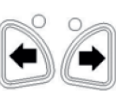
R-NET Quick Guide DK/UK - November 2010

QUICK GUIDE R-NET

UK GUIDE TO R-NET

 On/Off Button	The On/Off button turn the powerchair on/off. Do not use the On/Off button to stop the wheelchair unless there is an emergency. (If you do, you may shorten the life of the wheelchair drive components).
 Lights Button & LED*	This button activates and de-activates the wheelchair's lights. Depress the button to turn the lights on and depress the button again to turn them off. When activated the lights LED will illuminate.
 Horn Button	The Horn will sound while this button is depressed.
 Hazard button	This button activates and de-activates the wheelchair's hazard lights. Depress the button to turn the hazards button again to turn them off.
 Mode Button	The Mode button allows the user to navigate through the available operating Modes for the control system. The available modes are dependent on programming and the range of auxiliary output devices connected to the control system. - Move the joystick to the side to choose between the auxiliary output devices. When the device is found, move the joystick forward to activate the device.
 Profile Button	The Profile button allows the user to navigate through the available driving profiles for the control system. The number of available profiles is dependent on how the control system is programmed. It is possible to change the profiles, please contact a VELA technician or an authorised service technicians.
 Left & Right Indicator Button & LED*	This button activates and de-activates the wheelchair's left/right indicator. Depress the button to turn the indicator on and depress the button again to turn it off. When activated the indicator LED will flash in sync with the wheelchair's indicator(s).
 Speed Buttons Decrease/Increase	This button increases/decreases the driving speed.

* Lights button & Left/Right indicator is only available on the R-Net model xxxxxxxxxxxx

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