## Mavestøtte / Front support / Bauchplatte

### DANSK

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## Mavestøtte / Front support / Bauchplatte

### ENGLISH

The front support consists of an upholstered front pad, a right angle tube, and a double square-tube holder. See illustration 1. The front support is to be fitted to the VELA Miniwalk frame at the tube supporting the seat. To achieve this, the existing single square-tube holder attached to the seat is to be replaced by the double square-tube holder supplied. First, remove the rear stop by tilting it to its lowest position and subsequently loosening the Allen screw with a 6 mm Allen key as shown in illustration 2. Next, take off the seat by loosening the Allen screw on the underside of the square-tube holder located under the seat. Again, use a 6 mm Allen key. Separate the single square-tube holder from the seat by unscrewing the two Allen screws, using a 5 mm Allen key. Now the supplied double square-tube holder can be screwed onto the seat in place of the holder just removed. The double square-tube holder with the attached seat can now be placed on the VELA Miniwalk frame as shown in illustration 3. Using a 4 mm Allen key, tighten the Allen screw on the side of the upper tube of the double square-tube holder, thereby securing the seat unit to the frame of the VELA Miniwalk. Finally, insert the right angle tube with attached front pad into the lower tube of the double square-tube holder as shown in illustration 4. The position of the front pad can be adjusted back/forth as well as up/down. When a suitable position has been found, tighten the two Allen screws, i.e. the one on the side of the lower tube of the double square-tube holder and the one on the reverse side of the front pad - for both use a 4 mm Allen key.

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